HDL/LDL CHOLESTEROL CALIBRATOR

Calibratore per Colesterolo HDL/LDL
HDL/LDL-Cholesterol calibrator

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11/2008

PRODUCT CHARACTERISTICS

Human lyophilised serum.
The concentrations have been selected so as to ensure optimum calibration for use in manual and automatic analysers.

REAGENT

Lyophilized human pooled serum. Biological additives. Bacteriostatics agents.
The concentration of the components are lot-specific.

PRECAUTIONS

Human origin sera utilized have been tested and found negative for the presence of HBSAg, HCV and antibody to HIV (1/2). However handle cautiously as potentially infectious.

PREPARATION

- Carefully remove the plastic cap from the vial avoiding any loss of material.
- **Reconstitute (→) with 1.0 mL of distilled water.**
- Recap and gently mix by inverting the vial to dissolve the content. Make sure the powder is dislodged from the bottom of the vial.
- Return the vial to its upright position and let stand for at least 10 minutes before use (2 hours for LDL).

Improper handling and/or storage can affect results. Inaccurate reconstitution and errors in assay technique can cause erroneous results.

STORAGE AND STABILITY

The calibrator is stable until the expiration date on the label when stored at 2-10°C.
Store tightly capped when not use.

- After reconstitution is stable for:
7 days at 2-10 °C

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ASSIGNED VALUES OF ANALITES from LOT 61101

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Globe Diagnostics S.r.l.