**REF**

GD8469 00

# PLASMAPROTEIN MULTICALIBRATOR L MULTICALIBRATORE PLASMAPROTEINE L

**IVD****INTENDED USE**

Human serum containing plasmaprotein concentrations suitable to calibrate turbidimetric assays.

**REAGENT** – 2 vials containing 2 ml. Ready to use.  
Human serum. Sodium azide 0.95 g/l.

**STORAGE AND STABILITY**

The calibrator is stable up to the expiry date stated on the label, if stored at 2-8 °C. Do not use the calibrator after the expiry date. Keep the calibrator tightly closed to prevent contaminations during its use.

Presence of particles and turbidity is sign of deterioration.  
Opened vials are stable for 30 days at 2-8 °C.

**Note**

Bacterial contamination may cause reductions in the stability of many components of the calibrator.

**ASSIGNED VALUES**

The concentration values are traceable to the Standard Reference Material CRM 470 (Institute for Reference Materials and Measurements, IRMM).

**PRECAUTIONS IN USE**

All reagents contain inactive components such as preservatives (Sodium azide or others), surfactants etc. The total concentration of these components is lower than the limits reported by 67/548/ECC and 88/379/EEC directives about classification, packaging and labelling of dangerous substances. However, the reagents should be handled with caution, avoiding swallowing and contact with skin, eyes and mucous membranes.

The use of the laboratory reagents according to good laboratory practice is recommended.

The reagents from human donors have given negative results to anti-HIV 1/2, HBsAg and anti-HCV. It is recommended to handle with caution.

**Waste Management**

Please refer to local legal requirements.

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